INSET STANDARDIZED,
4" X 10"
0284301

Part number	72047UT11A
Drawing number	0284301
V-code (code 128 C)	2845

- > Artwork is actual size
> Drop template and notes before processing

2845

Refresh[®]

Contacts[™]

Eye Drops for All Contact Lens Wearers

Refresh[®] Contacts[™] provide instant soothing relief and long lasting comfort for all contact lens wearers. The unique formulation relieves the dryness and irritation caused by dust, glare, pollen, air pollution or over work giving more comfortable lens wear all day long.

Instructions for use:

- For extra comfort place 1 or 2 drops on each lens before inserting
- Use as required

Suitable for use with all types of contact lenses including disposables

Precautions:

- Do not use if allergic to any of the ingredients
- Do not swallow solution
- To avoid contamination, do not touch bottle tip on any surface, and avoid direct contact with eye
- Close the bottle tightly immediately after use
- Discard any remaining solution 60 days after opening
- Do not use after expiry date
- Keep out of the reach of children
- Store at room temperature

If you have continued problems please discuss with your lens care practitioner.

Contains:

15 mL solution

Sodium carboxymethylcellulose 0.5%

PURITE[™] 0.005%

Manufactured by:

Allergan Pharmaceuticals Ireland, Castlebar Road,
Westport, Co. Mayo, Ireland

0459

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REFRESH CONTACTS[®]

Contact Lens Care Sterile Solution

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Allergan Pharmaceuticals Ireland
Castlebar Road, Westport
County Mayo, Ireland

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