

## **INSTRUCTIONS FOR USE**

### **CELLUFRESH®**

#### **Lubricating Comfort Eye Drops**

Help ensure you always have comfortable lens wear all day long with CELLUFRESH. CELLUFRESH is a lubricating and moisturising eye drop that relieves discomfort for all contact lens wearers.

CELLUFRESH eye drops contain a unique combination of ingredients which provide instant refreshment and long-lasting comfort, helping to condition your lenses in the eye. CELLUFRESH may also be used by contact lens wearers to relieve eye irritation which may be caused by environmental factors such as dust, glare, pollen, air pollution or overwork. For additional comfort, you may apply a few drops to each lens before putting your lenses in the eye

#### **Description**

CELLUFRESH is a preservative free unit-dose sterile solution, which contains carboxymethylcellulose sodium 0,5 %, sodium chloride, sodium lactate, potassium chloride, calcium chloride dihydrate, magnesium chloride and purified water

#### **Instructions for use**

- Ensure that the container is intact.
- To open, completely twist off the tab.
- Place one or two drops into each affected eye
- Discard any remaining solution, and dispose of the container properly.
- For single use only.

#### **Precautions**

- Always use fresh solution.
- Do not use if the solution is cloudy or discoloured.
- To avoid contamination, do not touch dropper tip on any surface, and avoid direct contact with the eye.
- Do not use if allergic to any of the ingredients.
- Do not swallow the solution.
- Do not use after the expiry date.
- Keep out of the reach of children.
- Store at room temperature.
- Do not use if unit dose vial is damaged.
- Allow 10 minutes between the administrations of ophthalmic products.
- Ocular irritation may occur rarely.

### **Warnings**

This product is preservative free and should be used immediately after opening to avoid contamination which could result in eye infections.

If you have continued problems, please discuss with your eye care practitioner.

### **How supplied**

CELLUFRESH is supplied in foil wrapped pouches. Each vial containing 0,4 ml of solution.

### **Manufacturer**

Allergan Sales, LLC  
2525 Dupont Drive  
Irvine, CA 92612  
USA

Made in Ireland

### **Local Representative**

Allergan Pharmaceuticals (Pty) Ltd  
Allandale Building, Magwa Crescent  
Waterfall City, Midrand  
Johannesburg  
South Africa

Date of revision: March 2021

In case of an adverse event, please contact +27 11 545 6600 or send an email to [SA.Complaints@Allergan.com](mailto:SA.Complaints@Allergan.com)

