

INSTRUCTIONS FOR USE

OPTIVE FUSION™ UD

A combination formula for relief, comfort and protection from dry eye symptoms.

Preservative-Free

In **OPTIVE FUSION™ UD**, sodium carboxymethylcellulose (CMC) and sodium hyaluronate (HA) form a bridged matrix to synergistically relieve the symptoms of dry eye. HA is an effective ocular lubricant that helps support tear film stability.

Dual Action Lubricating and Osmoprotective Comfort Solution

OPTIVE FUSION™ UD is appropriate for use post LASIK surgery for relief of dryness and discomfort following surgery.

OPTIVE FUSION™ UD is compatible for use with all contact lenses.

Description

OPTIVE FUSION™ UD is a unit dose, preservative-free, sterile solution which contains sodium carboxymethylcellulose 0,5 %, sodium hyaluronate 0,1 %, glycerine 1,0 %, erythritol, levocarnitine, sodium lactate, potassium chloride, calcium chloride dihydrate, magnesium chloride hexahydrate and purified water.

Instructions for Use

Place 1 or 2 drops in the eye(s) as needed.

Precautions

- Do not use if allergic to any of the ingredients
- Do not swallow solution
- To avoid contamination or possible eye injury, do not touch vial tip on any surface, and avoid direct contact with the eye
- Do not use after expiry date marked on the product
- Keep out of the reach of children
- Store at room temperature
- Allow 5 minutes between the administrations of ophthalmic products
- Do not use if unit dose vial or product packaging is damaged
- Do not use if solution changes colour
- Keep away from sunlight
- Use vial once and discard

Warnings

Discontinue use and consult a doctor if you experience eye pain, changes in vision, continued redness or irritation of the eye, or if the condition worsens.

This product is preservative-free and should be used immediately after opening to avoid contamination, which could result in eye infections.

How Supplied

OPTIVE FUSION™ UD is supplied in foil wrapped pouches. Each vial contains 0,4 ml of solution.



Manufacturer

Allergan Pharmaceuticals Ireland
Castlebar Road
Westport, Co. Mayo
Ireland

Local representative

Allergan Pharmaceuticals (Pty) Ltd
30 New Road (entrance off Bavaria Road)
Randjespark Ext. 11, Midrand, 1682
Johannesburg, Gauteng
South Africa

