

INSTRUCTIONS FOR USE

OPTIVE FUSION™ UD

Preservative-Free

A unique combination formula for relief, comfort and protection from dry eye symptoms.

In OPTIVE FUSION™ UD, sodium carboxymethylcellulose (CMC) and sodium hyaluronate (HA) form a bridged matrix to synergistically relieve the symptoms of dry eye. HA is an effective ocular lubricant that helps support tear film stability.

Description

OPTIVE FUSION™ UD is a unit dose, preservative-free, sterile solution which contains sodium carboxymethylcellulose 0,5 %, sodium hyaluronate 0,1 %, glycerine 1,0 %, erythritol, levocarnitine, sodium lactate, potassium chloride, calcium chloride dihydrate, magnesium chloride hexahydrate and purified water.

OPTIVE FUSION™ UD is appropriate for use post LASIK surgery for relief of dryness and discomfort following surgery.

OPTIVE FUSION™ UD is compatible for use with all contact lenses.

Instructions for Use

Place 1 or 2 drops in the eye(s) as needed.

Precautions

- Do not use if allergic to any of the ingredients
- Do not swallow solution
- To avoid contamination or possible eye injury, do not touch vial tip on any surface, and avoid direct contact with the eye
- Do not use after expiry date marked on the product
- Keep out of the reach of children
- Store at room temperature
- Allow 5 minutes between the administrations of ophthalmic products
- Do not use if unit dose vial or product packaging is damaged
- Use vial once and discard
- Do not use if solution changes colour
- Keep away from sunlight

Warnings

This product is preservative-free and should be used immediately after opening to avoid contamination, which could result in eye infections.

Discontinue use and consult a doctor if you experience eye pain, changes in vision, continued redness or irritation of the eye, or if the condition worsens.

How Supplied

OPTIVE FUSION™ UD is supplied in foil wrapped pouches. Each vial contains 0,4 ml of solution.

Manufacturer

Allergan Sales, LLC
2525 Dupont Drive
Irvine, CA 92612
USA

Made in Ireland

Local Representative

Allergan Pharmaceuticals (Pty) Ltd
Allandale Building, Magwa Crescent
Waterfall City, Midrand
Johannesburg
South Africa

Date of revision: November 2020

In case of an adverse event, please contact +27 11 545 6600 or send an email to SA.Complaints@Allergan.com

