

## **INSTRUCTIONS FOR USE**

### **OPTIVE® GEL DROPS**

A unique gel drop formula for relief, comfort and sustained protection from dry eye symptoms. Long-lasting relief for night or day.

Dual Action Lubricating and Osmoprotective Comfort Solution

#### **Description**

OPTIVE® Gel Drops is a multi-dose sterile solution which contains sodium carboxymethylcellulose 1 %, glycerine 0,9 %, erythritol, levocarnitine, boric acid, sodium borate decahydrate, sodium citrate dihydrate, potassium chloride, calcium chloride dihydrate, magnesium chloride hexahydrate, purified water and is preserved with PURITE® 0,01 % which breaks down into natural tear components on the eye.

OPTIVE® Gel Drops is compatible for use with all contact lenses and can be used for 3 months from opening.

#### **Instructions for Use**

Place 1 or 2 drops in the eye(s) as needed.

#### **Precautions**

- Do not use if allergic to any of the ingredients
- Do not swallow solution
- To avoid contamination or possible eye injury, do not touch bottle tip on any surface, and avoid direct contact with the eye
- Close the bottle tightly immediately after use
- Discard any remaining solution 3 months after opening
- Product may cause transient blurring of vision
- Do not use after expiry date marked on the product
- Keep out of the reach of children
- Store at room temperature
- Do not use the bottle if the tamper-proof seal on the bottle neck is broken before you first use it
- Do not use if solution changes colour
- Allow 15 minutes between the administration of other ophthalmic products

#### **Warnings**

Discontinue use and consult a doctor if you experience eye pain, changes in vision, continued redness or irritation of the eye, or if the condition worsens.

### **How Supplied**

OPTIVE® Gel Drops is supplied in plastic bottles containing 2 ml or 10 ml

### **Manufacturer**

Allergan Sales, LLC  
2525 Dupont Drive  
Irvine, CA 92612  
USA

Made in USA

### **Local Representative**

Allergan Pharmaceuticals (Pty) Ltd  
Allandale Building, Magwa Crescent  
Waterfall City, Midrand  
Johannesburg  
South Africa

Date of revision: November 2020

In case of an adverse event, please contact +27 11 545 6600 or send an email to [SA\\_Complaints@Allergan.com](mailto:SA_Complaints@Allergan.com)

**STERILE A**