

---

## INSTRUCTIONS FOR USE

### OPTIVE OMEGA™ UD

#### Preservative-free

A unique combination formula containing linseed oil (a source of Omega-3) and osmoprotectants for rapid and long lasting relief of dry eye symptoms, and improvement in clinical signs of dry eye disease.

Suitable for use following LASIK refractive surgery.

#### Description

OPTIVE OMEGA™ UD is a unit dose, preservative-free, sterile solution which contains sodium carboxymethylcellulose 0,5 %, glycerine 1 %, polysorbate 80 0,5 %, linseed oil, castor oil, erythritol, levocarnitine, trehalose, and purified water.

OPTIVE OMEGA™ UD, is a unique combination formula containing linseed oil (a source of Omega-3), sodium carboxymethylcellulose (CMC) and osmoprotectants. OPTIVE OMEGA™ UD provides lubrication and moisture to the surface of the eye, helping to support tear film stability and restoring natural osmotic balance.

#### Instructions for Use

To open, twist and pull tab to remove. Instil 1 or 2 drops in the affected eye(s) as needed and discard container.

#### Precautions

- Do not use if allergic to any of the ingredients
- Do not swallow solution
- To avoid contamination or possible eye injury, do not touch vial tip on any surface, and avoid direct contact with the eye
- Do not use after expiry date marked on the product
- Keep out of the reach of children
- Store at room temperature
- Allow 5 minutes between the administration of ophthalmic products
- Do not use if unit dose vial or product packaging is damaged
- Do not use if solution changes colour
- Keep away from direct sunlight

#### Warnings

This product is preservative-free and should be used immediately after opening to avoid contamination, which could result in eye infections.

Discontinue use and consult a doctor if you experience eye pain, changes in vision, continued redness or irritation of the eye, or if the condition worsens.

### **How Supplied**

OPTIVE OMEGA™ UD is supplied in foil wrapped pouches. Each vial contains 0,4 ml of solution. Packs containing 2 vials are supplied without foil wrap.

### **Manufacturer**

Allergan Sales, LLC  
2525 Dupont Drive  
Irvine, CA 92612  
USA

Made in USA

### **Local Representative**

Allergan Pharmaceuticals (Pty) Ltd  
Allandale Building, Magwa Crescent  
Waterfall City, Midrand  
Johannesburg  
South Africa

Date of issue: November 2020

In case of an adverse event, please contact +27 11 545 6600 or send an email to [SA\\_Complaints@Allergan.com](mailto:SA_Complaints@Allergan.com)

