

INSTRUCTIONS FOR USE

OPTIVE PLUS™ UD

Preservative-Free

Triple-action lubricating and osmoprotective comfort solution.

OPTIVE PLUS UD has a unique triple-action formula that provides long lasting comfort. OPTIVE PLUS UD lubricates the surface of the eye and moisturises the ocular surface cells by restoring osmotic balance PLUS protecting your natural tears with a lipid enhancement.

Description

OPTIVE PLUS™ UD is a unit dose, preservative-free sterile solution which contains sodium carboxymethylcellulose 0,5 %, glycerine 1,0 %, castor oil 0,25 %, polysorbate 80 0,5 %, levocarnitine and erythritol.

Instructions for Use

Place 1 or 2 drops in the eye(s) as needed.

Precautions

- Do not use if allergic to any of the ingredients
- Do not swallow solution
- To avoid contamination or possible eye injury, do not touch vial tip on any surface, and avoid direct contact with the eye
- Do not use after expiry date marked on the product
- Keep out of the reach of children
- Store at room temperature
- Do not use if the unit dose vial is damaged
- Do not use if solution changes colour
- Allow 5 minutes between the administration of ophthalmic products
- Use vial once and discard

Warnings

Discontinue use and consult a doctor if you experience eye pain, changes in vision, continued redness or irritation of the eye, or if the condition worsens.

This product is preservative-free and should be used immediately after opening to avoid contamination which could result in eye infections.

How Supplied

OPTIVE PLUS UD is supplied in unit dose vials containing 0,4 ml

Manufacturer

Allergan Pharmaceuticals Ireland
Castlebar Road
Westport, Co. Mayo
Ireland

Made in U.S.A.

Local Representative

Allergan Pharmaceuticals (Pty) Ltd
Allandale Building, Magwa Crescent
Waterfall City, Midrand
Johannesburg
South Africa

Date of revision: July 2018

In case of an adverse event, please contact +27 11 545 6600 or send an email to SA_Complaints@Allergan.com

