

INSTRUCTIONS FOR USE

OPTIVE® UD

Preservative-free

Dual-action lubricating and osmoprotective comfort solution.

OPTIVE UD's dual-action formula provides long-lasting comfort. OPTIVE UD lubricates the surface of the eye and moisturises the ocular surface cells by restoring osmotic balance.

Description

OPTIVE® UD is a unit dose, preservative-free, sterile solution which contains sodium carboxymethylcellulose 0,5 %, glycerine 0,9 %, levocarnitine and erythritol.

OPTIVE® UD is appropriate for use post LASIK surgery for relief of dryness and discomfort following surgery.

OPTIVE UD is compatible for use with all contact lenses.

Instructions for Use

Place 1 or 2 drops in the eye(s) as needed.

Precautions

- Do not use if allergic to any of the ingredients
- Do not swallow solution
- Do not use after expiry date marked on the product
- Keep out of the reach of children
- Store at room temperature
- Allow 5 minutes between the administrations of ophthalmic products
- Do not use if unit dose vial is damaged
- To avoid contamination, do not touch vial tip on any surface, and avoid direct contact with the eye
- Use vial once and discard

Warnings

If you have continued problems please discuss with your eye care practitioner.

This product is preservative-free and should be used immediately after opening to avoid contamination which could result in eye infections.

How Supplied

OPTIVE UD comfort solution is supplied in unit dose vials containing 0,4 ml.

Manufacturer

Allergan Sales, LLC
2525 Dupont Drive
Irvine, CA 92612
USA

Made in Ireland

Local Representative

Allergan Pharmaceuticals (Pty) Ltd
Allandale Building, Magwa Crescent
Waterfall City, Midrand
Johannesburg
South Africa

Date of revision: November 2020

In case of an adverse event, please contact +27 11 545 6600 or send an email to
SA_Complaints@Allergan.com

